



# PROTECTION FROM

# COVID<sup>19</sup>

## RESPIRATOR with protection class FFP3

- Made with **five layers** of nonwoven material, including two separate layers of meltblown nonwoven fabric comprising the filtration material. Total weight: 160 g/m<sup>2</sup>
- Light, breathable and hypoallergenic material **with a high filtration efficiency of ≥99%**
- The comfortable integrated nosepiece and soft elastic band ensure they are comfortable to wear and form a perfect seal, **gentle on the skin**
- Easy to breathe thanks to low pressure resistance, without an exhalation valve



**FFP3 - protection class  
NR - only for single use**



### Instructions for putting on the respirator:

1. Remove the respirator from the package, open it and check to make sure it is in good condition. Place the respirator on your chin. The nosepiece should be at the top.
2. Pull the upper elastic over your head and place it on the back of your head.
3. Pull the bottom elastic over your head and place it at the back of your neck. Adjust both the elastics so that the respirator fits comfortably.
4. Adjust the nosepiece with your fingers along the bridge of the nose, so that the respirator fits properly. Always make sure the respirator forms a proper seal. If air escapes around the nose or along the sides of the respirator, adjust the nosepiece or placement of the respirator.



**Detailed instructions for use can be found inside the package.**

**REF: 1320100799**

Pack of 50 respirators

Personal protective equipment cat. III pursuant to Directive EU 2016/425  
Fulfilment of all requirements of standard EN 149: 2001 + AC: 2009