

TOP GLOVE SDN. BHD.

INSTRUCTION FOR USE

NITRILE EXAM POWDERED GLOVES

Sphere of use : Powdered nitrile examination gloves are worn to protect the hand of users from chemical and also to prevent transmission of diseases.

Instructions : **Caution:-** Users should consider the circumstances of use in deciding whether to remove residue powder on gloves after donning. Powder can be removed by wiping gloves thoroughly with a sterile wet sponge or sterile wet towel or other effective methods.

After use wearers should visually check the glove and remove any contamination from the outer surface before removing the gloves from the hands. Alternatively carefully peel the gloves off the hands so that the contaminated glove outer does not touch your skin.

: Storage:- Store in a cool dry place. Avoid excessive heat. Opened box should be shielded from exposure to direct sun or fluorescent lighting.

Shelf Life : 5 years upon the manufacturing date.

NITRILE EXAM POWDER-FREE GLOVES

Sphere of use : Powder-free nitrile examination glove are worn to protect the hands of users from chemical and also to prevent transmission of diseases.

Instructions : **Caution:-** After use wearers should visually check the gloves and remove any contamination from the outer surface before removing the gloves from the hand. Alternatively carefully peel the gloves off the hand so that the contaminated glove outer does not touch your skin.

: Storage:- Store in a cool dry place. Avoid excessive heat. Opened box should be shielded from exposure to direct sun or fluorescent lighting.

Shelf Life : 5 years upon the manufacturing date.

Prepared by:
Regulatory Affairs Unit

Verified by:
Pn Noor Akilah Saidin
Deputy General Manager, RA