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Intended use

The device is intended for use in patients after removing the dialysis line or any kind of injection or blood sampling, in order to maintain pressure on the site involved and prevent blood leakage and internal subcutaneous spillage by temporarily occluding the hole made.

CAUTIONS

Before use, check the integrity of the device.

The device is intended for use by qualified personnel.

Do not use the device without supervision or contaminated.

Use only until achieving hemostasis, do not exceed application time > 20 minutes.

Instruction of use

a) Hold the external device holder by thumb, the inside handle with fingers.

b) Extend the handles - apply slight pressure on the handle of the device.

c) Protect the injection site with a sterile dressing, set up the device at the injection site.

d) Slowly release and close the unit, making sure that compression is applied to the injection site.

Cleaning and disinfection

The device is reusable.

The unit must always be washed and disinfected after use and before reuse.

The device can be washed and disinfected by immersing it in a disinfectant to the tools (solution-benzalkonium chloride, sodium hypochlorite, chlorhexidine digluconate, or alcohol-based), not containing solutions of acids and corrosive agents. In any case, follow the instructions and dose of disinfectant.

Warning

Before using the device, disinfect the injection site and secure sterile dressings for example sterile gauze swab .

Never place the device in direct contact with the wound.

Do not use for compression of cuts, lacerations, for purposes other than intended.

Wear protective gloves to prevent biological risks during use and risks as a result of contact with biological fluids or other substances that can be dangerous.

Do not use the device if it is contaminated with body fluids, blood and / or other factors.

Contraindications and side effects

Using the device does not cause any side effects.

There were no reported disturbance or threat of use associated with other devices.

Disposal

The device should be disposed of as hospital medical waste potentially contaminated.

Storage conditions

Store it in a dry and clean place away from direct heat sources.

Replace the unit if it is damaged or if its essential parts showing signs of wear or if the spring has lost its functionality.

NOTE: It is important to read the instructions for proper use of the device.

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