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EN - INSTRUCTIONS

Product Name:
Sterile paraffin gauze dressings

Brief introduction & Intended Use:

Paraffin gauze dressingsare mainly used for burns orscalds. It helps for smoothing and protecting the wound and allowing free passage of exudate.

Components:

Medical absorbent gauze,Petrolatum oil.Isolation membrane

Product Performance:

- This product is non-irritating to the skin, non-cytotoxic, and has no delayedhypersensitivity.

- The gauze is made of cotton interwoven base fabric, which is moist and has no odor. The gauze is attached with PET on both sides.
- Paraffinis fully adsorbed and uniform in thickness.
- This product has been sterilized by radiation sterilization.

Function and usage of paraffin gauze dressings:

- Sterile paraffin gauze dressings are made of medical absorbent gauze, petroleum jelly and barrier film. Medical absorbent gauze has good absorbency and can be better wrapped with petroleum jelly. Paraffinoil has a certain degree of moisturization, which can reduce the risk of wound adhesion, and can effectively prevent wound impregnation and promote wound healing. The barrier prevents oil from oozing out, which better locks the Paraffinoil.
- Usage:
 - According to the size of the wound, choose a sterile paraffin gauze dressingthat is more than 2 cm from the edge of the wound.
 - Open the outer packaging and remove the sterile paraffin gauze dressingsaseptically.
 - Cover the paraffingauze on the wound after debridement.
 - Select a two-layer fixed dressing to fix.

Note:

- Sterile paraffin gauze dressings are single use products and must not be reused. Damage to the inner packaging is prohibited. Do not use it if primary packaging is damaged, opened, in this case sterilization is not effective.
- Do not use expired products.
- Accumulated use of this product shall not exceed 30 days.

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